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Instruction for immediate life-saving  
measures in case of accidents

COCHE  
CAR

MOTOCICLETA  
MOTORCYCLE

Download this document in additional  
languages by scanning the QR code

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## A Giving help and reassurance, emergency call

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It is a moral obligation of every human being to help victims of accidents. Give first aid and care as good as possible.

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Have the rescue services called out as soon as possible.

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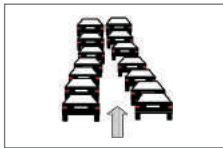
### Emergency call

Where?	Give exact details about the place of emergency
What happened?	Give a short description of the accident scene
What number?	Tell how many persons are injured
What kind of injuries?	Specify if there are any threatening injuries
Wait	The operator will finish the call when he has all informations he needs

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## C How to act at the site of an accident

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Allow ambulances to pass between the waiting vehicles.

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First of all safeguard the site of the accident. Put up a warning triangle.

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Assess the situation and act cautiously and considerately.

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Address casualty, open the safety belt and grasp his/her hips turn casualty's back towards you.

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Get a hold of casualty's forearm passing your arms under his armpits. Drag casualty out of the vehicle and lay him down at a safe place.

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Open the visor and address casualty. If not responsive: see **G** Recovery position

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Remove crash helmet by two first aiders as shown here.

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Check breathing. If casualty is breathing: see **G** Recovery Position

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Patient does not respond: see **G** Unconsciousness

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No breath to be heard, no movement of the chest:  
see **H** Respiratory standstill

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No signs of life like breathing, swallowing, coughing: see **I**  
Circulatory standstill

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## G Unconsciousness, recovery position

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Open the visor and address casualty. If not responsive: Recovery position

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**Recovery position:** Stretch near side arm and push it under the hip.

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Get ahold of far side shoulder and hip and turn casualty cautiously on his side.

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Recline the head, open the mouth, and bend backward arm.

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Keep casualty warm and call for the ambulance.

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In case of respiratory failure, artificial respiration:

1. Recline head
2. Keep nose or mouth closed
3. Give breath into mouth or nose
4. Turn head and watch casualty's expiration

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Circulatory failure: Cardio-pulmonary resuscitation **I** + **H**

**Cardio-pulmonary resuscitation:** Continue CPR until a doctor is available, applying artifical breathing and chest compression (100 / min) at a rate of 2:15.

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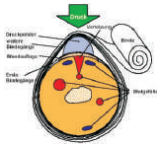
In case of severe bleedings from the arm, first elevate arm and apply indirect pressure at the upper arm.

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Dress severe bleedings from arm or leg by applying a pressure bandage as shown here.

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Casualties who have a shock are pale, they are cold and they have a feeble pulse.

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Place casualty with his legs in an elevated position, reassure him, and keep him warm. There is a rescue blanket in the first aid kit.

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In case of injuries or illnesses in the abdominal region, place casualty with bent knees for relaxation as shown here.

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In case of breathing difficulties or cardiac infarct, place casualty with head and chest elevated if he is responsive.

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In case of head injuries such as a concussion of the brain, place casualty with his head elevated if he is responsive.

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A casualty who has a shock is positioned on his back with his legs elevated if he is responsive.

# N Índice DIN 13164 y 13167

## N Table of contents DIN 13164 and DIN 13167

Descripción	Description	DIN 13164	DIN 13167
Rollo esparadrapo DIN 13019-A, 5 m × 2,5 cm	Adhesive tape DIN 13019-A, 5 m × 2.5 cm	1 ×	1 ×
Herida esparadrapos: <ul style="list-style-type: none"><li>• 4 × Yeso para heridas DIN 13019-E, 6 × 10 cm</li><li>• 2 × Vendaje para los dedos, 2 × 12 cm</li><li>• 2 × Vendaje de la punta de los dedos</li><li>• 4 × Yeso para heridas, 2,5 × 7,2 cm</li><li>• 2 × Yeso para heridas, 1,9 × 7,2 cm</li></ul>	Wound plaster set: <ul style="list-style-type: none"><li>• 4 × wound plaster DIN 13019-E, 6 × 10 cm</li><li>• 2 × finger dressing, 2 × 12 cm</li><li>• 2 × fingertip dressing</li><li>• 4 × wound plaster, 2.5 × 7.2 cm</li><li>• 2 × wound plaster, 1.9 × 7.2 cm</li></ul>	1 ×	1 ×
Paquete de vendas DIN 13151-K, estéril	First aid dressing DIN 13151-K, sterile	1 ×	
Paquete de vendas DIN 13151-M, estéril simple	First aid dressing DIN 13151-M, single sterile	2 ×	1 ×
Paquete de vendas DIN 13151-G, estéril	First aid dressing DIN 13151-G, sterile	1 ×	1 ×
Paño de vendaje DIN 13152-BR, 40 × 60 cm, estéril	Bandaging cloth DIN 13152-BR, 40 × 60 cm, sterile	1 ×	
Paño de vendaje DIN 13152-A, 80 × 60 cm, estéril	Bandaging cloth DIN 13152-A, 80 × 60 cm, sterile	1 ×	1 ×
Compresas de heridas, 10 × 10 cm, estéril	Wound dressings (compresses), 10 × 10 cm, sterile	6 ×	
Venda elástico DIN 61634-FB6, 4 m × 6 cm	Elastic bandage DIN 61634-FB6, 4 m × 6 cm	2 ×	
Venda elástico DIN 61634-FB8, 4 m × 8 cm	Elastic bandage DIN 61634-FB8, 4 m × 8 cm	3 ×	
Venda triangulara DIN 13168-D, vellón	Triangular bandage DIN 13168-D fleece	2 ×	
Manta de salvamento, 210 × 160 cm, plata/oro	Emergency blanket, 210 × 160 cm, silver/gold	1 ×	1 ×
Tijeras DIN 58279-A 145	Scissors DIN 58279-A 145	1 ×	1 ×
Bolsa de 4 guantes desechables DIN EN455, vinyle	Bag of 4 disposable gloves DIN EN455, vinyl	1 ×	1 ×
Toallitas húmedas 12,5 × 18 cm	Cleansing wipe 12.5 × 18 cm	2 ×	2 ×
Instrucciones de primeros auxilios para accidentes con tabla de contenido	First aid instructions for accidents with table of contents	1 ×	1 ×

**FLEXEO GmbH**

Im Maisel 17

65232 Taunusstein

Germany

[info@flexeo.shop](mailto:info@flexeo.shop)

[www.flexeo.shop](http://www.flexeo.shop)

**Download this document in additional  
languages by scanning the QR code**

**or visit [www.flexeo.shop/manual/](http://www.flexeo.shop/manual/)**