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Instruction for immediate life-saving
measures in case of accidents

AUTO
CAR

MOTO
MOTORCYCLE

Download this document in additional
languages by scanning the QR code

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A Giving help and reassurance, emergency call

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It is a moral obligation of every human being to help victims of accidents. Give first aid and care as good as possible.

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Have the rescue services called out as soon as possible.

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Emergency call

Where?

Give exact details about the place of emergency

What happened?

Give a short description of the accident scene

What number?

Tell how many persons are injured

What kind of injuries?

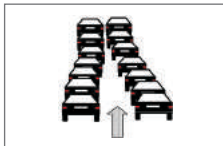
Specify if there are any threatening injuries

Wait

The operator will finish the call when he has all informations he needs

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C How to act at the site of an accident

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Allow ambulances to pass between the waiting vehicles.

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First of all safeguard the site of the accident. Put up a warning triangle.

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Assess the situation and act cautiously and considerately.

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Address casualty, open the safety belt and grasp his/her hips turn casualty's back towards you.

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Get a hold of casualty's forearm passing your arms under his armpits.
Drag casualty out of the vehicle and lay him down at a safe place.

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Open the visor and address casualty. If not responsive: see **G** Recovery position

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Remove crash helmet by two first aiders as shown here.

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Check breathing. If casualty is breathing: see **G** Recovery Position

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Patient does not respond: see **G** Unconsciousness

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No breath to be heard, no movement of the chest:
see **H** Respiratory standstill

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No signs of life like breathing, swallowing, coughing: see **I** Circulatory standstill

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G Unconsciousness, recovery position

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Open the visor and address casualty. If not responsive: Recovery position

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Recovery position: Stretch near side arm and push it under the hip.

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Get ahold of far side shoulder and hip and turn casualty cautiously on hide side.

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Recline the head, open the mouth, and bend backward arm.

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Keep casualty warm and call for the ambulance.

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In case of respiratory failure, artificial respiration:

1. Recline head
 2. Keep nose or mouth closed
 3. Give breath into mouth or nose
 4. Turn head and watch casualty's expiration
- Give 15 breaths per minute.

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Circulatory failure: Cardio-pulmonary resuscitation **I** + **H**

Cardio-pulmonary resuscitation: Continue CPR until a doctor is available, applying artificial breathing and chest compression (100 / min) at a rate of 2:15.

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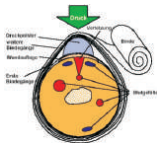
In case of severe bleedings from the arm, first elevate arm and apply indirect pressure at the upper arm.

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Dress severe bleedings from arm or leg by applying a pressure bandage as shown here.

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Casualties who have a shock are pale, they are cold and they have a feeble pulse.

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Place casualty with his legs in an elevated position, reassure him, and keep him warm. There is a rescue blanket in the first aid kit.

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In case of injuries or illnesses in the abdominal region, place casualty with bent knees for relaxation as shown here.

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In case of breathing difficulties or cardiac infarct, place casualty with head and chest elevated if he is responsive.

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In case of head injuries such as a concussion of the brain, place casualty with his head elevated if he is responsive.

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A casualty who has a shock is positioned on his back with his legs elevated if he is responsive.

N Table des matières DIN 13164 et DIN 13167

N Table of contents DIN 13164 and DIN 13167

Description	Description	DIN 13164	DIN 13167
Sparadrap DIN 13019-A, 5 m × 2,5 cm	Adhesive tape DIN 13019-A, 5 m × 2.5 cm	1 ×	1 ×
Pansements adhésifs: • 4 × Plâtre de la plaie DIN 13019-E, 6 × 10 cm • 2 × L'habillage des doigts, 2 × 12 cm • 2 × L'habillage du bout des doigts • 4 × Plâtre de la plaie, 2,5 × 7,2 cm • 2 × Plâtre de la plaie, 1,9 × 7,2 cm	Wound plaster set: • 4 × wound plaster DIN 13019-E, 6 × 10 cm • 2 × finger dressing, 2 × 12 cm • 2 × fingertip dressing • 4 × wound plaster, 2.5 × 7.2 cm • 2 × wound plaster, 1.9 × 7.2 cm	1 ×	1 ×
Cartouche de pansement DIN 13151-K, stérile	First aid dressing DIN 13151-K, sterile	1 ×	
Cartouche de pansement DIN 13151-M, stérile séparée	First aid dressing DIN 13151-M, single sterile	2 ×	1 ×
Cartouche de pansement DIN 13151-G, stérile	First aid dressing DIN 13151-G, sterile	1 ×	1 ×
Pansement pour brûlures DIN 13152-BR, 40 × 60 cm, stérile	Bandaging cloth DIN 13152-BR, 40 × 60 cm, sterile	1 ×	
Pansement pour brûlures DIN 13152-A, 80 × 60 cm, stérile	Bandaging cloth DIN 13152-A, 80 × 60 cm, sterile	1 ×	1 ×
Plaie compresses, 10 × 10 cm, stérile	Wound dressings (compresses), 10 × 10 cm, sterile	6 ×	
Bande extensible DIN 61634-FB6, 4 m × 6 cm	Elastic bandage DIN 61634-FB6, 4 m × 6 cm	2 ×	
Bande extensible DIN 61634-FB8, 4 m × 8 cm	Elastic bandage DIN 61634-FB8, 4 m × 8 cm	3 ×	
Bande triangulaire DIN 13168-D, non-tissé	Triangular bandage DIN 13168-D fleece	2 ×	
Couverture de survie, 210 × 160 cm, argentine/dorée	Emergency blanket, 210 × 160 cm, silver/gold	1 ×	1 ×
Ciseaux DIN 58279-A 145	Scissors DIN 58279-A 145	1 ×	1 ×
Sac à 4 gants à usage unique DIN EN455, vinyle	Bag of 4 disposable gloves DIN EN455, vinyl	1 ×	1 ×
Lingettes nettoyantes 12,5 × 18 cm	Cleansing wipe 12.5 × 18 cm	2 ×	2 ×
Directive pour les premiers soins avec table des matières	First aid instructions for accidents with table of contents	1 ×	1 ×

FLEXEO GmbH

Im Maisel 17

65232 Taunusstein

Germany

info@flexeo.shop

www.flexeo.shop

**Download this document in additional
languages by scanning the QR code**

or visit www.flexeo.shop/manual/