FLEXEO HOT OR COLD COMPRESS

IMPORTANT

Read the instructions carefully before use.

USE HOT TO HELP RELIEF

- Aches, pain and itching • Pre/post workout • Swelling & sprains
- Sinus discomfort • Arthritis • Sore muscles

USE COLD TO HELP RELIEF

- Swelling & sprains • Toothache & headache • Aches, pain & itching • Pain after surgery • Pre/post workout • Arthritis • Fever

COOLING INSTRUCTIONS

Place the compress in the freezer or refrigerator until the desired temperature is reached. Wrap the compress with a cloth if it is too cold.

HEATING INSTRUCTIONS

A. In hot water

Put the compress in hot water (max. 80 °C) until the desired temperature is reached.

B. Microwave

Wrap the compress with a moist cloth and place it in the microwave. Set the power below 800 watts and heat in 10-second intervals to reach the desired temperature. Check all parts of the pack before reheating so that one part of the pack does not get too hot. Knead the pack to make sure the heat is evenly distributed. Turn the pack 180 degrees to reheat in the microwave. **DO NOT OVERHEAT!**

WARNINGS

- Take care when removing from the microwave oven, the pack may be very hot.
- To avoid frostbite or burns, set it aside until it reaches a temperature your skin can tolerate.
- Apply for not more than 20 minutes at a time, with 20 minutes breaks between applications.
- **DO NOT** wear it while sleeping. Check skin frequently during use.
- **DO NOT** use it on people unable to remove the product, including children, infants, and elderly or on people unable to follow all use instructions.
- **DO NOT** overheat.
- **DO NOT** place extra pressure over the product.
- **DO NOT** swallow. Keep out of reach of children and pets.
- **DO NOT** use hot cold pack contents leak and/or wrap is damaged or torn. If hot cold pack contents come in contact with your eyes, remove wrap, rinse affected area with water and seek professional assistance immediately.
- Stop use and ask a doctor if pain worsens or persists for more than 7 days, or if you have any discomfort, burning swelling, rash or other changes in your skin.
- Ask a doctor before use if you have diabetes, poor circulation or heart disease, rheumatoid arthritis or if you are pregnant.

NOTICE

Seal the pack in the resalable bag and store in the freezer to make the life term longer. It would also be more convenient for next cold therapy. Wipe clean with a damp cloth and allow to dry. The pack is not machine washable. Discard if any signs of damage, tears or holes. Check for leaks before use. Overheating may cause the gel pack to rupture or leak. If the gel pack begins to expand while heating in the microwave, stop heating and let the pack cool. Cleaning: Wash with warm soapy water after use.

DISPOSAL METHOD

This material is not hazardous waste, please dispose of the pack in accordance with local regulations.

CAUTION

1. When using as a heat treatment, be careful to use a moderate heat for a limited time to avoid burns.
2. When using as a cooling treatment, always wrap the pack in towel/cloth to avoid potential frostbite.
3. Never leave the gel pack on for extended periods of time or wear a pack while sleeping.
4. For external use only.
5. Do not puncture the gel pack or use if punctured.
6. If content is exposed to the eye and skin, rise with water and seek medical advice.

FLEXEO

Importer: **FLEXEO GmbH**

Im Maisel 17, 65232 Taunusstein, Germany
info@flexeo.shop, www.flexeo.shop

Wuhan Pakcare Bio-technology Co.,Ltd
No.28-1 North Linkong Road,
Hengdian Street, Huangpi District,
Wuhan City, China, 430301

SUNGO Europe B.V., Olympisch
Stadion 24, 1076DE, Amsterdam,
Netherlands

Download this document in additional languages by scanning the QR code

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Importeur: **FLEXEO GmbH**