# FLEXEO HOT OR COLD COMPRESS

## IMPORTANT

Read the instructions carefully before use.

## USE HOT TO HELP RELIEF

- Aches, pain and itching • Pre/post workout • Swelling & sprains
- Sinus discomfort • Arthritis • Sore muscles

## USE COLD TO HELP RELIEF

- Swelling & sprains • Toothache & headache • Aches, pain & itching
- Pain after surgery • Pre/post workout • Arthritis • Fever

## COOLING INSTRUCTIONS

1. Place the pack in freezer or refrigerator for at least two hours or until desired temperature is achieved.
2. Before application, check to make sure the temperature is suitable.
3. If it feels well, you can place the pack directly on your skin. Otherwise wrap the pack in towel/cloth before using.

## HEATING INSTRUCTIONS

1. Start with a room temperature pack.
2. Wrap the pack in moist cloth or paper towel and place pack in the center of clean microwave turntable (not touching the sides).
3. Notes: Rotate 180 degrees halfway through heating time and make sure that any additional microwave features are turned off, such as "browning".
4. Microwave for time indicated below. The product size is indicated on the package.
5. If more heat required, reheat for additional time indicated below intervals until safe desired temperature is reached for comfortable application. Before reheating, check if the temperature of any parts of the pack is too high (normally it occurs at the edge of the pack) and knead the pack to make the temperature even, then rotate 180 degrees of the pack and reheat in microwave.
6. Always checking the pack is at the correct temperature before applying. If it feels well, you can place the pack directly on your skin. Otherwise wrap for safety in a towel of cloth before using. Overheating is not allowed. For maximum total heating time see template below.

Compress size	Rating	Heating time for room temperature	Reheating time if more heat required	Maximum total heating time
30.5 × 20.3 cm	800 watts	30 seconds	10 seconds	50 seconds
	1,000 watts	20 seconds	5 seconds	50 seconds
27.4 × 12.7 cm	800 watts	30 seconds	10 seconds	50 seconds
	1,000 watts	20 seconds	5 seconds	50 seconds
19 × 7.5 cm	800 watts	10 seconds	5 seconds	20 seconds
	1,000 watts	10 seconds	5 seconds	20 seconds
14 × 13 cm	800 watts	15 seconds	5 seconds	30 seconds
	1,000 watts	10 seconds	5 seconds	30 seconds

## WARNINGS

- Take care when removing from the microwave oven, the pack may be very hot.
- To avoid frostbite or burns, set it aside until it reaches a temperature your skin can tolerate.
- Apply for not more than 20 minutes at a time, with 20 minutes breaks between applications.
- DO NOT wear it while sleeping. Check skin frequently during use.

- DO NOT use it on people unable to remove the product, including children, infants, and elderly or on people unable to follow all use instructions.
- DO NOT overheat.
- DO NOT place extra pressure over the product.
- DO NOT swallow. Keep out of reach of children and pets.
- DO NOT use hot cold pack contents leak and/or wrap is damaged or torn. If hot cold pack contents come in contact with your eyes, remove wrap, rinse affected area with water and seek professional assistance immediately.
- Stop use and ask a doctor if pain worsens or persists for more than 7 days, or if you have any discomfort, burning swelling, rash or other changes in your skin.
- Ask a doctor before use if you have diabetes, poor circulation or heart disease, rheumatoid arthritis or if you are pregnant.

## NOTICE

Seal the pack in the resalable bag and store in the freezer to make the life term longer. It would also be more convenient for next cold therapy. Wipe clean with a damp cloth and allow to dry. The pack is not machine washable. Discard if any signs of damage, tears or holes. Check for leaks before use. Overheating may cause the gel pack to rupture or leak. If the gel pack begins to expand while heating in the microwave, stop heating and let the pack cool.

## DISPOSAL METHOD

This material is not hazardous waste, please dispose of the pack in accordance with local regulations.

## CAUTION

When using as a heat treatment, be careful to use a moderate heat for a limited time to avoid burns.

When using as a cooling treatment, always wrap the pack in towel/cloth to avoid potential frostbite.

Never leave the gel pack on for extended periods of time or wear a pack while sleeping.

For external use only.

Do not puncture the gel pack or use if punctured.

If content is exposed to the eye and skin, rise with water and seek medical advice.

Download this document in additional languages by scanning the QR code

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Importeur: **FLEXEO GmbH**
Wuhan Pakcare Bio-Technology Co., Ltd.,
SUNGO Europe B.V., Olympisch Stadion 24,